

PREMIUM

8.0
8.0T

Tcoil



DIGITAL

Hearing INSTRUMENTS



USER GUIDE

Content

Welcome	5
<hr/>	
Your hearing instruments	7
<hr/>	
Instrument type	7
Getting to know your hearing instruments	9
Components and names	10
Controls	16
Hearing programs	18
Features	19
Batteries	20
<hr/>	
Battery size and handling tips	20
Replacing batteries	22

Daily use	24
Turning on and off	24
Inserting and removing the hearing instruments	26
Adjusting the volume	33
Changing the hearing program	34
Special listening situations	35
Audio streaming with iPhone	35
On the phone	35
Airplane mode	37
Audio induction loops (optional)	38
Maintenance and care	40
Hearing instruments	40
Ear pieces	42
Professional maintenance	47

Further information	48
Safety information	48
Accessories	48
Symbols used in this document	49
Troubleshooting	50
Conformance information	53
Service and warranty	54

Welcome

Thank you for choosing our hearing instruments to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This guide, along with support from your Hearing Care Professional, will help you understand the advantages and greater quality of life your hearing instruments offer.

To receive the most benefit from your hearing instruments, it is recommended that you wear them every day, all day. This will help you get adjusted to them.



CAUTION

It is important to read this user guide and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

For more information, please visit the Kirkland Signature 8.0 User Portal at KS8userportal.com

Your hearing instruments



This user guide describes optional features that your hearing instruments may or may not have.

Ask your Hearing Care Professional to indicate the features that are valid for your hearing instruments.

Instrument type

Your hearing instruments are a RIC (Receiver-in-Canal) model. The receiver is placed within the ear canal and connected to the instrument via a receiver cable. The instruments are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.

The wireless functionality enables advanced audiological features and synchronization between your two hearing instruments.

Your hearing instruments feature Bluetooth® low energy* technology allowing for easy data exchange with your smartphone, and for seamless audio streaming with your iPhone**.

* The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc., and any use of such marks by the legal manufacturer of this product is under licenses. Other trademarks and trade names are those of their respective owners.

** iPad, iPhone, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.

Getting to know your hearing instruments

It is recommended to familiarize yourself with your new hearing instruments. With the instruments in your hand try using the controls and note their location on the instrument. This will make it easier for you to feel and press the controls while wearing the hearing instruments.




If you have problems pressing the controls of your hearing instruments while wearing them, you can use a remote control to control your instruments. With a dedicated smartphone app you have even more control options.

Components and names

This user guide describes several types of hearing instruments. Use the following pictures to identify the type of hearing instrument that you use.

KS 8.0

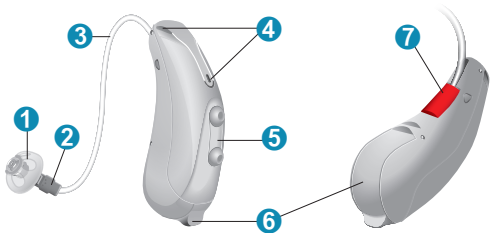

-
- 1 Ear piece
 - 2 Receiver
 - 3 Receiver cable
 - 4 Microphone openings
 - 5 Rocker switch (control)
 - 6 Battery compartment (on/off switch)
 - 7 Side indicator (red = right ear, blue = left ear) and receiver connection
-

KS 8.0 T



-
- 1 Ear piece
 - 2 Receiver
 - 3 Receiver cable
 - 4 Microphone openings
 - 5 Rocker switch (control)
 - 6 Battery compartment (on/off switch)
 - 7 Side indicator (red = right ear, blue = left ear) and receiver connection
-

You can use the following standard ear pieces:

Standard ear pieces	Size
----------------------------	-------------



Click Sleeve
(vented or closed)



Click Dome™ single
(open or closed)



Click Dome semi-open



Click Dome double

You can easily exchange the standard ear pieces. Read more in section "Maintenance and care".

Custom-made ear pieces




Custom shell



Click Mold™

Controls

With the rocker switch you can, for example, adjust the volume or switch hearing programs. Your Hearing Care Professional has programmed your desired functions to the rocker switch.

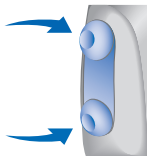

Rocker switch function

L **R**

Press briefly:

Program up/down

Volume up/down

TV stream on/off

Rocker switch function	L	R
<hr/> Press for about 2 seconds:		
Program up/down	<input type="checkbox"/>	<input type="checkbox"/>
Volume up/down	<input type="checkbox"/>	<input type="checkbox"/>
TV stream on/off	<input type="checkbox"/>	<input type="checkbox"/>
<hr/> Press for more than 3 seconds:		
Standby/turn on	<input type="checkbox"/>	<input type="checkbox"/>
Activate/deactivate airplane mode	<input type="checkbox"/>	<input type="checkbox"/>

L = Left, R = Right



You can also use a remote control to change hearing programs and adjust the volume or your hearing instruments. With our smartphone app you have even more control options.

Hearing programs

1

2

3

4

5

6

Read more in section "Changing the hearing program".

Features

- Power-on delay** enables whistle-free insertion of hearing instruments.

Read more in section "Turning on and off".

- A **telecoil** is built into the instrument, so it can connect with audio induction loops.

Read more in section "Audio induction loops".

Batteries

When the battery is low the sound becomes weaker or you will hear an alert signal. The battery type will determine how long you have to replace the battery.

Battery size and handling tips

Ask your Hearing Care Professional for recommended batteries.

Battery size:

312


13

- Always use the correct battery size for your hearing instruments.
- Remove the batteries if you intend not to use the hearing instruments for several days.
- Always carry spare batteries.
- Remove empty batteries immediately and follow your local recycling rules for battery disposal.

Replacing batteries

Removing the battery:

- ▶ Open the battery compartment.
- ▶ Use the magnet stick to pull out the battery. The magnet stick is included in your package.

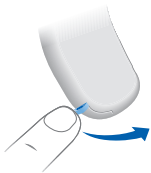

Inserting the battery:

- ▶ If the battery has a protective film, remove it only when you are ready to use the battery.

- ▶ Insert the battery with the "+" symbol facing up (refer to the picture).

- ▶ Carefully close the battery compartment. If you feel resistance, the battery is not inserted correctly. Do not attempt to close the battery compartment by force. It can be damaged.



Daily use

Turning on and off

You have the following options to turn your hearing instruments on or off.

Via battery compartment:

- ▶ Turning on: Close the battery compartment.
The **default** volume and hearing program are set.
- ▶ Turning off: Open the battery compartment to the first catch.

Via rocker switch:

- ▶ Turning on or off: Press and hold the rocker switch for several seconds. Refer to section "Controls" for the settings of the rocker switch.

After turning on, the **previously** used volume and hearing program remain set.

When wearing the hearing instruments, an alert tone can indicate when an instrument is being turned on or off.

When the **power-on delay** is activated, the hearing instruments turn on after a delay of several seconds. During this time you can insert the hearing instruments into your ears without experiencing unpleasant feedback whistling.

The "power-on delay" can be activated by your Hearing Care Professional.

Inserting and removing the hearing instruments

Your hearing instruments have been fine-tuned for your right and left ear. Colored markers indicate the side:


- red marker = right ear
- blue marker = left ear




Inserting a hearing instrument:

- ▶ For Click Sleeves, take care that the bend of the Click Sleeve is in line with the bend of the receiver cable.


Correct:

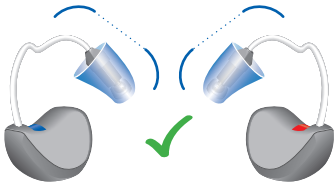

Incorrect:

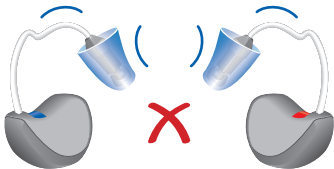

- ▶ Hold the receiver cable at the bend closer to the ear piece.
- ▶ Carefully push the ear piece in the ear canal ①.
- ▶ Twist it slightly until it sits well.
Open and close your mouth to avoid accumulation of air in the ear canal.
- ▶ Lift the hearing instrument and slide it over the top of your ear ②.

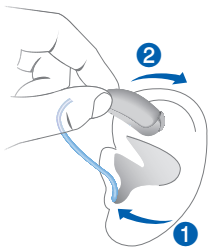



CAUTION

Risk of injury!


- ▶ Insert the ear piece carefully and not too deeply into the ear.



- It may be helpful to insert the **right** hearing instrument with the **right** hand and the **left** hearing instrument with the **left** hand.
- If you have problems inserting the ear piece, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the ear piece.


The optional retention strand helps to securely retain the ear piece in your ear. To position the retention strand:

- ▶ Bend the retention strand and position it carefully into the bottom of the bowl of your ear (refer to the picture).

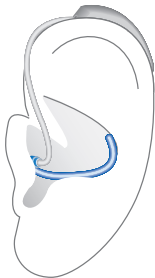

Removing a hearing instrument:

- ▶ Lift the hearing instrument and slide it over the top of your ear **1**.
- ▶ If your hearing instrument is equipped with a custom shell or a Click Mold, remove it by pulling the small removal cord toward the back of your head.
- ▶ For all other ear pieces: Grip the receiver in the ear canal with two fingers and pull it carefully out **2**. Do not pull the receiver cable.

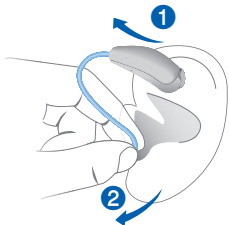



CAUTION

Risk of injury!

- ▶ In very rare cases the ear piece could remain in your ear when removing the hearing instrument. If this happens, have the ear piece removed by a medical professional.

Clean and dry your hearing instruments after usage. Read more in section "Maintenance and care".

Adjusting the volume

Your hearing instruments automatically adjust the volume to the listening situation.

- ▶ If you prefer manual volume adjustment, press the rocker switch.

Refer to section "Controls" for the setting of the rocker switch.

An optional signal tone can indicate the volume change. Once the maximum or minimum volume has been reached, you may hear an optional signal tone.

Changing the hearing program

Depending on the listening situation, your hearing instruments automatically adjust their sound.

Your hearing instruments may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- ▶ To change the hearing program, press the rocker switch. Refer to section "Controls" for the setting of the rocker switch. Refer to section "Hearing programs" for a list of your hearing programs.

The default volume is applied.

Special listening situations

Audio streaming with iPhone

Your hearing instruments are **Made for iPhone** hearing instruments. This means you can get phone calls and listen to music from your iPhone directly into your hearing instruments.

For more information about compatible iOS devices, pairing, audio streaming and other useful functions, refer to the Bluetooth/App card included in delivery.

On the phone

When you are on the phone, hold the telephone receiver slightly above your ear. The hearing instrument and


telephone receiver have to be aligned. Turn the receiver slightly so that the ear is not completely covered.

Telephone program

You may prefer a certain volume when using the phone. Ask your Hearing Care Professional to configure a telephone program.

- ▶ Switch to the telephone program whenever you are on the phone.

If a telephone program is configured for your hearing instruments, it is listed in section "Hearing programs".

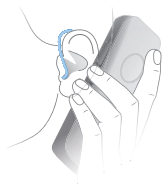

Airplane mode

In areas where the use of Bluetooth technology is not allowed (e.g. on some airplanes), you can activate the airplane mode. This turns Bluetooth in your hearing instruments off temporarily. The hearing instruments still work without Bluetooth, however direct audio streaming will not be possible and other functions will not be available.

- ▶ To activate or deactivate the airplane mode, use the smartphone app or use the controls of your hearing instruments.

Refer to section "Controls" for more information.

An alert tone indicates when airplane mode is activated or deactivated.

Audio induction loops (optional)

Some telephones and public places like theatres offer the audio signal (music and speech) via an audio induction loop. With this system, your hearing instruments can directly receive the desired signal – without distracting noise in the environment.

You can usually recognize audio induction loop systems by certain signs.



Whether a telecoil is built into your hearing instrument is listed in section "Features". If a telecoil is built in, you have the following options.

Ask your Hearing Care Professional to configure a telecoil program.

- ▶ Switch to the telecoil program whenever you are in a place with an audio induction loop.

If a telecoil program is configured for your hearing instruments, it is listed in section "Hearing programs".

Maintenance and care

To prevent damage it is important that you take care of your hearing instruments and follow a few basic rules, which will soon become a part of your daily routine.

Hearing instruments

Drying and storage

- ▶ Dry your hearing instruments overnight.
- ▶ Ask your Hearing Care Professional for recommended drying products.
- ▶ For longer periods of non-use, store your hearing instruments with open battery compartment and batteries removed in a drying system to avoid the adverse effects of moisture.

Cleaning

Your hearing instruments have a protective coating. However, if not cleaned regularly you may damage the hearing instruments or cause personal injury.

- ▶ Clean your hearing instruments daily with a soft, dry tissue.
- ▶ Never use running water or immerse the devices in water.
- ▶ Never apply pressure while cleaning.
- ▶ Ask your Hearing Care Professional for recommended cleaning products, special care sets, or more information on how to keep your hearing instruments in good condition.



Ear pieces


Cleaning

Cerumen (ear wax) may accumulate on the ear pieces. This may affect sound quality. Clean the ear pieces daily.

For all types of ear pieces:

- ▶ Clean the ear piece right after removal with a soft and dry tissue.

This prevents that cerumen becomes dry and hard.

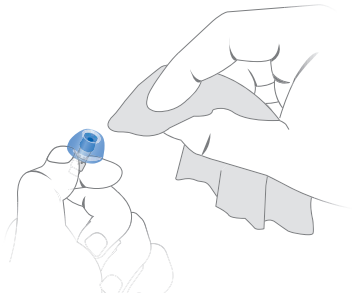

For Click Domes or Click Sleeves:

- ▶ Squeeze the tip of the ear piece.




Exchanging

Exchange standard ear pieces approximately every three months. Replace them sooner if you notice cracks or other changes. The procedure for exchanging standard ear pieces depends on the type of ear piece. In section "Components and names", your Hearing Care Professional has marked your type of ear piece.

Exchanging Click Domes

- ▶ To remove the old Click Dome, use the removal tool or follow the instructions on the packaging of the Click Domes. Be careful not to pull the receiver cable.

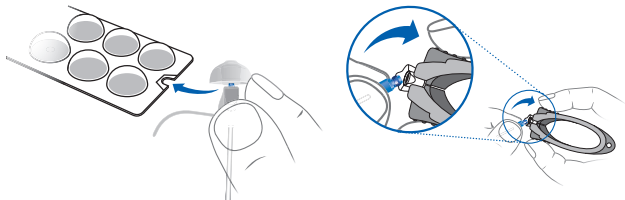

The removal tool is available as an accessory.

- ▶ Take particular care that the new Click Dome clicks noticeable in place.
- ▶ Check the firm fit of the new Click Dome.



Exchanging Click Sleeves

- ▶ Turn the Click Sleeve inside out.
- ▶ Grasp the Click Sleeve and the receiver (not the receiver cable) with your fingers and pull the Click Sleeve off.
- ▶ Take particular care that the new Click Sleeve clicks noticeably in place.



Professional maintenance

Your Hearing Care Professional can perform a thorough professional cleaning and maintenance.

Custom-made ear pieces and wax filters should be exchanged as needed by your Hearing Care Professional.

Ask your Hearing Care Professional for your individual recommendation of maintenance intervals and for support.

Further information

Safety information

For more safety information, refer to the safety manual supplied with the device.

Accessories

You can use the Smart Direct App for convenient remote control of your hearing instruments. The app also allows you to deactivate signal tones, to create individual sound profiles and many more functions.

You can also use a remote control.

Consult your Hearing Care Professional for further information.

Symbols used in this document



Points out a situation that could lead to serious, moderate, or minor injuries.



Advice and tips on how to handle your device better.

Made for



iPod



iPhone



iPad

“Made for iPod”, “Made for iPhone” and “Made for iPad” mean that an electronic accessory has been designed to connect specifically to iPod, iPhone, or iPad, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this

accessory with iPod, iPhone, or iPad may affect wireless performance.

Troubleshooting

Problem and possible solutions

Sound is weak.

- Increase the volume.
 - Replace the empty battery.
 - Clean or exchange the ear piece.
-

Problem and possible solutions

Hearing instrument emits whistling sounds.

- Reinsert the ear piece until it fits securely.
 - Decrease the volume.
 - Clean or exchange the ear piece.
-

Sound is distorted.

- Decrease the volume.
 - Replace the empty battery.
 - Clean or exchange the ear piece.
-

Hearing instrument emits signal tones.

- Replace the empty battery.
-

Problem and possible solutions

Hearing instrument does not work.

- Turn the hearing instrument on.
 - Gently close the battery compartment completely.
 - Replace the empty battery.
 - Make sure the battery is inserted correctly.
 - Power-on delay is active. Wait several seconds and check again.
-

Consult your Hearing Care Professional if you encounter further problems.

Conformance information

USA and Canada

Certification information for hearing instruments with wireless functionality:

KS 8.0

- FCC ID: SGI-RIC002
- IC: 267AB-RIC002

KS 8.0 T

- Contains FCC ID: SGI-RFM001, SGI-MIM001
- IC: 267AB-WP4N1

Service and warranty

Serial numbers

Instrument

Receiver unit

Left:

Right:

Service dates

1:

4:

2:

5:

3:

6:


Warranty

Date of purchase:

Warranty

Warranty period in months:

Your Hearing Care Professional

 **Legal Manufacturer**
Sivantos GmbH
Henri-Dunant-Strasse 100
91058 Erlangen
Germany
Phone +49 9131 308 0

Document No. 03357-99T01-7600 EN-USA
Order/Item No. 10954801
Master Rev01, 01.2018
© 01.2018, Sivantos GmbH.
All rights reserved



DISTRIBUTED BY:
COSTCO WHOLESALE CORPORATION
P.O. BOX 34535
SEATTLE, WA 98124-1535
USA
1-800-774-2678
WWW.COSTCO.COM

DISTRIBUÉ PAR :
COSTCO FRANCE
1 AVENUE DE BRÉHAT
91140 VILLEBON-SUR-YVETTE
FRANCE
01 80 45 01 10 • WWW.COSTCO.FR

IMPORTADO POR:
COSTCO WHOLESALE SPAIN S.L.U.
POLÍGONO EMPRESARIAL LOS GAVILANES
C/ AGUSTÍN DE BETANCOURT, 17
28906 GETAFE (MADRID) ESPAÑA
NIF: B86509460
900 111 155 • WWW.COSTCO.ES

DISTRIBUTED BY:
COSTCO WHOLESALE UK LTD /
COSTCO ONLINE UK LTD
HARTSPRING LANE
WATFORD, HERTS
WD25 8JS • UNITED KINGDOM
01923 213113 • WWW.COSTCO.CO.UK

DISTRIBUTED BY:
COSTCO WHOLESALE ICELAND EHF.
KAUPTÚN 3-7, 210 GARDABAER
ICELAND
WWW.COSTCO.IS

IMPORTED BY:
COSTCO WHOLESALE
AUSTRALIA PTY LTD
17-21 PARRAMATTA ROAD
LIDCOMBE NSW 2141
AUSTRALIA
WWW.COSTCO.COM.AU

IMPORTED BY/ IMPORTE PAR : COSTCO WHOLESALE CANADA LTD.
415 W. HUNT CLUB ROAD, OTTAWA, ONTARIO
1-800-463-3783 • K2E 1 CS, CANADA • WWW.COSTCO.CA

IMPORTED AND DISTRIBUTED BY: COSTCO WHOLESALE KOREA, LTD.
40, ILJIK-RO, GWANGMYEONG-SI, GYEONGGI-DO, 14347, KOREA
1899-9900 • WWW.COSTCO.CO.KR

IMPORTED BY / MANUFACTURED FOR: COSTCO PRESIDENT TAIWAN, INC.
NO. 656 CHUNG-HWA 5TH ROAD, KAOHSIUNG, TAIWAN
COMPANY TAX ID: 96972798
449-9909 OR 02-449-9909 (IF CELLPHONE) • WWW.COSTCO.COM.TW

IMPORTED BY: COSTCO WHOLESALE JAPAN LTD.
3-1-4 IKEGAMI-SHINCHO, KAWASAKI-KU, KAWASAKI-SHI, KANAGAWA
210-0832 JAPAN • 044-281-2600 • WWW.COSTCO.CO.JP

18KS0258m



If you're not completely satisfied with this Kirkland Signature product, your money will be refunded.
Si ce produit Kirkland Signature ne vous donne pas entière satisfaction, nous vous le rembourserons.
Si no queda completamente satisfecho con su producto Kirkland Signature, se le devolverá el dinero.
For UK only: This does not affect your statutory rights.
Pour le Royaume-Uni seulement : ceci n'affecte pas vos droits légaux.
Para Reino Unido solamente: Esto no afecta sus derechos legales.