

Inserting & removing your hearing aids

1. Insert Batteries In Your Hearing Aids

- Open the battery compartment
- Insert a new battery
- "+" side facing up (flat side)
- Make sure the battery "+" and compartment "+" match.




2. Turn Your Hearing Aids On

- Close the battery door
- Your hearing aids may turn on immediately or your Hearing Professional may have programmed a 6, 12, or 18 second start-up delay

- No Delay 12 Second
- 6 Second 18 Second

3. Insert Your Hearing Aids

- A: Grasp receiver/dome between your thumb and forefinger
- Insert dome into your ear canal
 - Gently work dome into proper position by slightly adjusting it until it is firmly seated in your ear
- B: Lift the hearing aid over the top of your ear
- Adjust it to fit behind your ear
- C: Press dome and receiver wire slightly for a secure and comfortable fit.
- Place retention strand into the bottom "bowl" of your ear (if applicable)

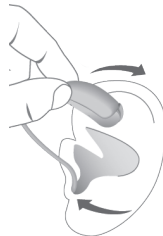

Rocker switch functions



This is how your hearing aid rocker switches are programmed to function:

ROCKER SWITCH FUNCTION	PRESS ROCKER SWITCH		
	Short 1 second	Medium 2 seconds	Long 3 seconds
	L R	L R	L R
Program Up/Down	<input type="checkbox"/> <input type="checkbox"/>	<input type="checkbox"/> <input type="checkbox"/>	
Volume Up/Down	<input type="checkbox"/> <input type="checkbox"/>	<input type="checkbox"/> <input type="checkbox"/>	
TV Stream*	<input type="checkbox"/> <input type="checkbox"/>	<input type="checkbox"/> <input type="checkbox"/>	
Power On/Off			<input type="checkbox"/> <input type="checkbox"/>
Airplane Mode			<input type="checkbox"/> <input type="checkbox"/>

*Requires accessory Smart Transmitter 2.4

These are the signal tones for your hearing aid:

SIGNAL SOUNDS	L	R
Program Change	<input type="checkbox"/>	<input type="checkbox"/>
Volume Change	<input type="checkbox"/>	<input type="checkbox"/>
Volume Limit	<input type="checkbox"/>	<input type="checkbox"/>
Volume On Position	<input type="checkbox"/>	<input type="checkbox"/>
Low Battery	<input type="checkbox"/>	<input type="checkbox"/>
Power On/Off	<input type="checkbox"/>	<input type="checkbox"/>
Bluetooth® On/Off	<input type="checkbox"/>	<input type="checkbox"/>

Your hearing aid programs

This is how your hearing aid memories are programmed:

PROGRAM	PROGRAM NUMBER
<input type="checkbox"/> Automatic Adjusts to all environments for maximum speech clarity	<input checked="" type="checkbox"/> 1
<input type="checkbox"/> TV Streams TV directly to your hearing aids when used with the Smart Transmitter 2.4*	<input type="checkbox"/>
<input type="checkbox"/> Noise/Party For crowded places with lots of background noise	<input type="checkbox"/>
<input type="checkbox"/> iFocus 360 Lets you hear people talking beside or behind you without turning your head	<input type="checkbox"/>
<input type="checkbox"/> Music Enhances a variety of music for a full, rich sound quality	<input type="checkbox"/>
<input type="checkbox"/> Reverb Reducer Reduces echo in reverberant environments	<input type="checkbox"/>
<input type="checkbox"/> Outdoors Hear sounds from all around and reduces wind noise	<input type="checkbox"/>
<input type="checkbox"/> XPhone Phone calls are automatically heard in both ears when a phone is held up to one ear	<input type="checkbox"/>
<input type="checkbox"/> Telecoil/Loop ¹ Amplifies signals from public loop systems	<input type="checkbox"/>
<input type="checkbox"/> Other	<input type="checkbox"/>

*Requires accessory Smart Transmitter 2.4



¹ Kirkland Signature 8.0 T only

Direct Streaming with iPhone

You can stream telephone calls, music, and other audio directly to your hearing aids. First, you need to pair your hearing aids to your iPhone one time.

From your iPhone home screen tap on:

- Settings
- General
- Accessibility
- MFi Hearing Devices
- Turn on Bluetooth
- Open and close hearing aid battery doors
- Tap hearing aids on phone screen
- Pair to each hearing aid
- Tap on your connected hearing aids to:
 - Increase/decrease volume
 - Change programs
 - Live Listen (turn your phone into a remote mic)

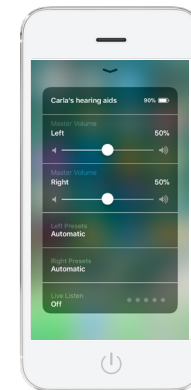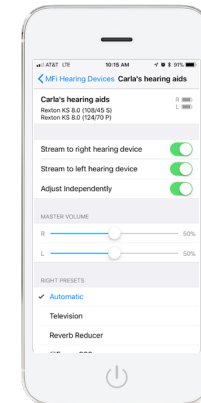

Quickly access these controls from your iPhone by simply triple clicking your Home button and select **Hearing Devices**.



iPad, iPhone, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.


Connex Smart Direct App for iPhone

Control your hearing aids from your smart phone discreetly with the Connex Smart Direct app.

1. Go to the App Store
2. Search for Connex Smart Direct
3. Download and install app
4. Follow instructions to pair hearing aids to phone



- Direct Stream phone calls and audio from apps to Kirkland Signature 8.0
- Motion Sense adjusts hearing aid directionality when moving for maximum awareness of your surroundings
- Change hearing aid functions discreetly from your phone:
 - Volume up/down
 - Program change
 - Microphone-pattern adjustments
 - Battery status
 - Sound balance
 - Remote mic (Live Listen)
 - Environment analysis
 - System sounds

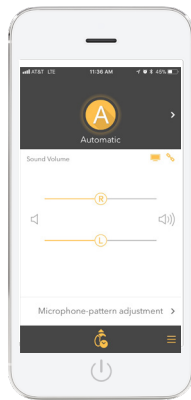


Apple App Store is a trademark of Apple Inc.


Connex Smart Direct App for Android

Control your hearing aids from your smart phone discreetly with the Connex Smart Direct app.

1. Go to the Play Store
2. Search for Connex Smart Direct
3. Download and install app
4. Follow instructions to pair hearing aids to phone

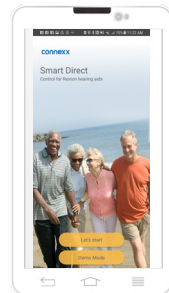

- Motion Sense adjusts hearing aid directionality when moving for maximum awareness of your surroundings
- Change hearing aid functions discreetly from your phone:
 - Volume up/down
 - Program change
 - Microphone-pattern adjustments
 - Battery status
 - Sound balance
 - Remote mic (Live Listen)
 - Environment analysis
 - System sounds

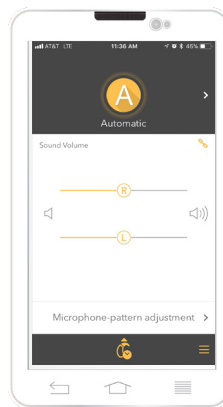

Android and Google Play are trademarks of Google Inc.

Troubleshooting

Problem & Possible Solutions
Sound is weak:
Increase the volume
Replace the battery
Clean or exchange the ear piece
Hearing aid emits whistling sounds:
Reinsert the ear piece until it fits securely
Decrease the volume
Clean or exchange the ear piece
Sound is distorted:
Decrease the volume
Replace the battery
Clean or exchange the ear piece
Hearing instrument emits spontaneous signal tones:
Replace the battery
Hearing aid does not work:
Turn the hearing aid on
Gently close the battery compartment completely
Replace the battery
Make sure the battery is inserted correctly
Power-on-delay is active. Wait several seconds and check again

Consult your Hearing Care Professional if you have further problems.



DIGITAL
Hearing
INSTRUMENTS



Quick Start Guide

PREMIUM

8.0
8.0T

Tcoil

THIS IS NOT A SUBSTITUTION FOR READING YOUR OWNER'S MANUAL. FOR MORE IN DEPTH INSTRUCTIONS, PLEASE REFER TO YOUR OWNER'S MANUAL.