

LOADING SETTINGS TO REXTON HEARING AIDS

LOADING SETTINGS TO BOTH HEARING AIDS

1. Open member's Noah session to be restored to the hearing aids (Fig. 1)
2. Turn hearing aids off/on. This activates the 2 minute connection window
3. Click on Connect Noahlink Wireless (bottom left corner in Connexx) (Fig. 2)
4. Fitting will load and is ready for programming
5. Click X at top right in Connexx to close and save session (Fig. 3)

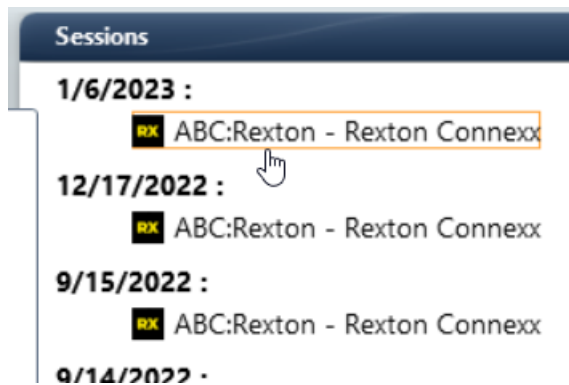


Fig. 1

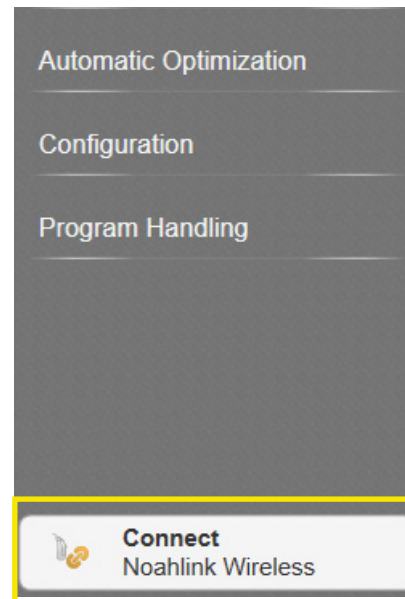


Fig. 2

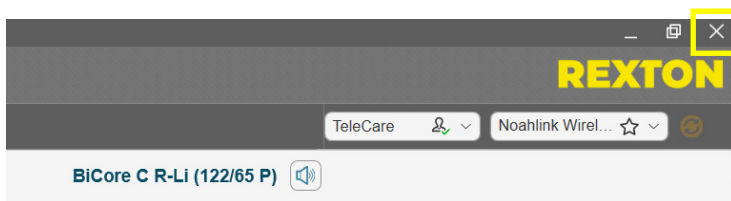


Fig. 3

LOADING SETTINGS TO REXTON HEARING AIDS

LOADING SETTINGS TO 1 HEARING AID ONLY

1. Open member's Noah session to be restored to the hearing aid (Fig. 1)
2. Turn hearing aids off/on. This activates the 2 minute connection window
3. Select Service Program Right or Program Left (Fig. 2)
4. Select the hearing aid from the list and click Ok. Hearing aids will be programmed and remain in Simulation mode. (Fig. 3)
5. Click X at top right in Connexx to close and save session (Fig. 4)

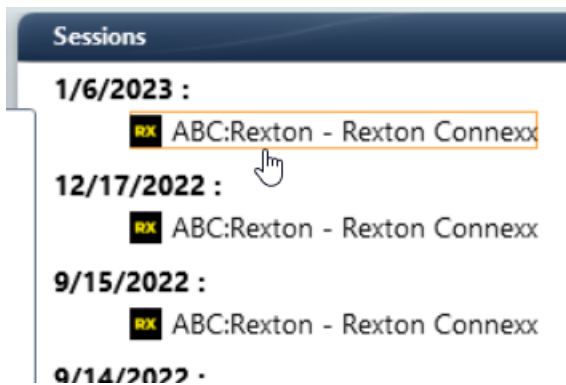


Fig. 1

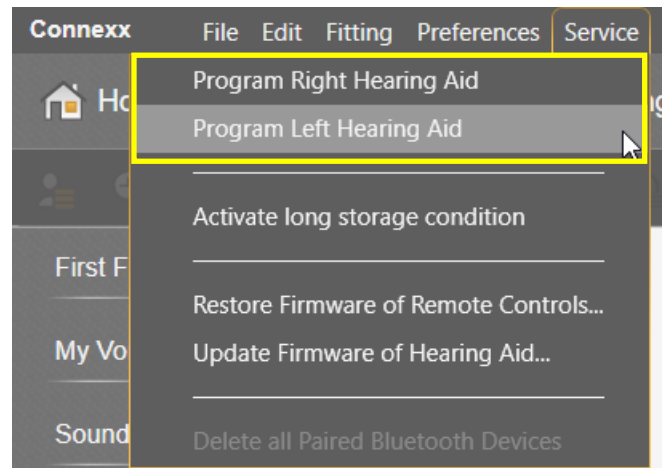


Fig. 2

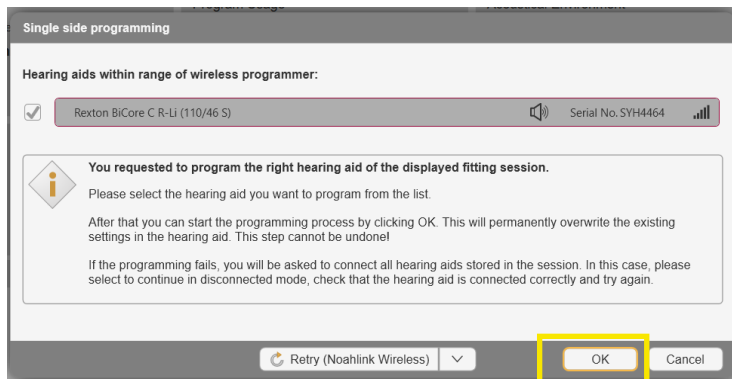


Fig. 3

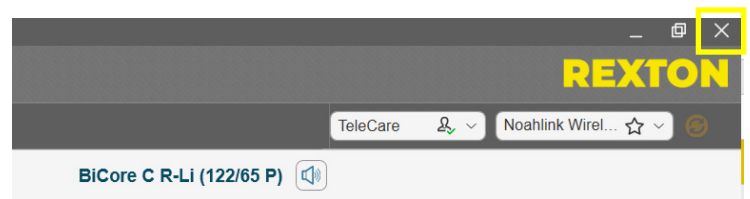


Fig. 4