BiCore B-Li M Rugged

USER GUIDE 2
(INSTRUCTION FOR USE)

Made for
 iPhone | iPad | iPod

REXTON

CAUTION

It is important to read this user guide and the user guide 1 (safety manual) thoroughly and completely. Follow the safety information to avoid damage or injury.

CONTENT

Welcome	5
Your hearing aids	6
Instrument type	6
Getting to know your hearing aids	7
Components and names	7
Controls	9
Hearing programs	10
Features	10
Daily use	11
Charging	11
Turning on and off	12
Switching to standby mode	13
Inserting and removing the hearing aids	14
Adjusting the volume	19
Changing the hearing program	19
Further adjustments (optional)	20
Special listening situations	21
On the phone	21
Audio streaming with iPhone	21
Audio streaming with Android phone	22
Bluetooth	22
Audio induction loop	22

Maintenance and care	24
Hearing aids	24
Ear pieces and tubes	27
Professional maintenance	30
Important safety information	31
Further information	34
Safety information	34
Accessories	34
Operating, transport, and storage conditions	34
Disposal information	36
Symbols used in this document	36
Troubleshooting	37
Country-specific information	38
Service and warranty	39

WELCOME

Thank you for choosing our hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This guide, along with support from your Hearing Care Professional, will help you understand the advantages and greater quality of life your hearing aids offer.

To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

CAUTION

It is important to read this user guide and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

Before using your hearing aids for the first time, fully charge them. Follow the instructions in the charger's user guide.

YOUR HEARING AIDS

This user guide describes optional features that your hearing aids may or may not have.

Ask your Hearing Care Professional to indicate the features that are valid for your hearing aids.

INSTRUMENT TYPE

Your hearing aids are BTE (Behind-The-Ear) models. A tube delivers the sound from the hearing aid into the ear. The instruments are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.

A power cell (lithium-ion rechargeable battery) is permanently built into your hearing aid. This enables easy charging with Charging Station B-M.

The wireless functionality enables advanced audiological features and synchronization between your two hearing aids.

Your hearing aids feature Bluetooth® low energy* technology allowing for easy data exchange with your smartphone, and for seamless audio streaming with your iPhone**.

* The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc., and any use of such marks by WS Audiology Denmark A/S of this product is under licenses. Other trademarks and trade names are those of their respective owners.

** iPad, iPhone, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.

GETTING TO KNOW YOUR HEARING AIDS

It is recommended to familiarize yourself with your new hearing aids. With the instruments in your hand try using the controls and note their location on the instrument. This will make it easier for you to feel and press the controls while wearing the hearing aids.

i If you have problems pressing the controls of your hearing aids while wearing them, you can ask your Hearing Care Professional whether a remote control or a smartphone app to control your instruments is available.

COMPONENTS AND NAMES

Your hearing aids are either attached to an earhook and a custom-made ear piece or to a standard tube (ThinTube) and a standard ear piece.

-
- | | |
|-----------------------|--|
| 1 Ear piece | 4 Rocker switch (control, on/off switch) |
| 2 Tube (ThinTube) | 5 Earhook |
| 3 Microphone openings | 6 Side indication |
-

You can use standard ear pieces together with a standard tube or custom-made ear pieces together with an earhook and tube.

You can easily exchange the standard ear pieces and clean the standard tube. Read more in section "Maintenance and care".

Standard ear pieces / tubes		Size
<input type="checkbox"/>		Instant Open
<input type="checkbox"/>		Instant Tulip
<input type="checkbox"/>	ThinTube	

Custom-made ear pieces	
<input type="checkbox"/>	Examples:
	

CONTROLS

With the rocker switch you can, for example, switch hearing programs. The Hearing Care Professional has programmed your desired functions to the rocker switch.

Rocker switch function	L	R
Press briefly:		
Program up/down	<input type="checkbox"/>	<input type="checkbox"/>
Volume up/down	<input type="checkbox"/>	<input type="checkbox"/>
Tinnitus Function level up/down	<input type="checkbox"/>	<input type="checkbox"/>
TV stream on/off	<input type="checkbox"/>	<input type="checkbox"/>
Press for about 2 seconds:		
Program up/down	<input type="checkbox"/>	<input type="checkbox"/>
Volume up/down	<input type="checkbox"/>	<input type="checkbox"/>
Tinnitus Function level up/down	<input type="checkbox"/>	<input type="checkbox"/>
TV stream on/off	<input type="checkbox"/>	<input type="checkbox"/>
Press for more than 3 seconds:		
Turn on/off	<input type="checkbox"/>	<input type="checkbox"/>

L = Left, R = Right

You can also use a remote control to change hearing programs and adjust the volume of your hearing aids. With our smartphone app you have even more control options.

HEARING PROGRAMS

1

2

3

4

5

6

Read more in section "Changing the hearing program".

FEATURES

The **Tinnitus Function** generates a sound to distract you from your tinnitus.

DAILY USE

To assure comfortable and easy use of your hearing aids they are equipped with an onboard control element. In addition we offer an app for Android as well as iPhones which makes handling your devices even easier. Contact your Hearing Care Professional for downloading and installing the smartphone app.

CHARGING

Place your hearing aids in the charger.

- ▶ Follow the instructions in the charger's user guide. You can also find useful tips for charging in the charger's user guide.

LOW POWER INDICATION

You will hear an alert signal to indicate the power cell is running low. The signal will be repeated every 20 minutes. Depending on the usage of your hearing aids, you have approximately 1.5 hours to charge your hearing aids before they stop working.

Note that the power cell runtime is reduced if too many features are used simultaneously.

TURNING ON AND OFF

You have the following options to turn your hearing aids on or off.

Via charger:

- ▶ Turning on: Take the instruments out of the charger.
The startup melody is played in your hearing aids. The **default** volume and hearing program are set.
- ▶ Turning off: Place the hearing aids in the charger.

Note that the charger must be connected to the power supply. For details refer to the charger's user guide.

Via rocker switch:

- ▶ Turning on: Press and hold the lower part of the rocker switch until the startup melody starts to play. Release the rocker switch while the melody is playing.

The **default** volume and hearing program are set.

- ▶ Turning off: Press and hold upper or lower part of the rocker switch for several seconds. A power down melody is played.

See also the tips for charging in the charger's user guide.

SWITCHING TO STANDBY MODE

Via remote control you can switch your hearing aids to standby mode. This mutes the hearing aids. When you leave standby mode, the **previously** used volume and hearing program are set.

Note:

- In standby mode, the hearing aids are **not** turned off completely. They draw some power.

Therefore we recommend to use standby mode only for a short period of time.

- If you want to leave standby mode, but the remote control is not at hand: Turn your hearing aids off and on again (via rocker switch or by shortly placing them in the charger until green LEDs light up). In this case, the **default** volume and hearing program are set.

INSERTING AND REMOVING THE HEARING AIDS


Your hearing aids have been fine-tuned for your right and left ear. Colored markers indicate the side:

- red marker = right ear
- blue marker = left ear

Inserting a hearing aid:

- ▶ Hold the tube close to the ear piece.
- ▶ Carefully push the ear piece into the ear canal **1**.
- ▶ Twist it slightly until it sits well.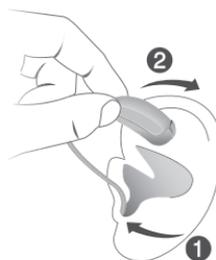

Open and close your mouth to avoid accumulation of air in the ear canal.

- ▶ Lift the hearing aid and slide it over the top of your ear **2**.

CAUTION

Risk of injury!

- ▶ Always wear the tube with an ear piece.
- ▶ Make sure that the ear piece is completely attached.

CAUTION

Risk of injury!

- ▶ Insert the ear piece carefully and not too deeply into the ear.


- It may be helpful to insert the **right** hearing aid with the **right** hand and the **left** hearing aid with the **left** hand.
- If you have problems inserting the ear piece, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the ear piece.

The optional retention strand helps to securely retain the ear piece in your ear. To position the retention strand:

Concha Lock Sleeve 3.0

Slide over as shown in the image.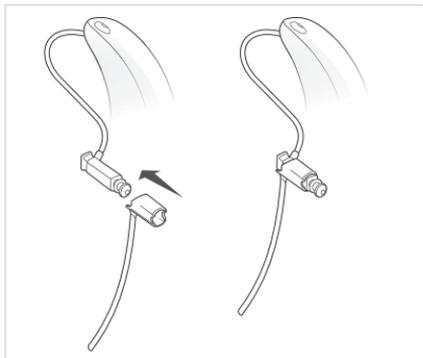

Concha Lock 3.0

Align the end of the retention strand with the flat end of the receiver. Then press the retention strand towards the receiver until it clicks into place.


- ▶ Bend the retention strand and position it carefully into the bottom of the bowl of your ear (refer to the picture).

Removing a hearing aid:

- ▶ Lift the hearing aid and slide it over the top of your ear ①.
- ▶ Hold the tube close to the ear piece and pull the ear piece out carefully ②.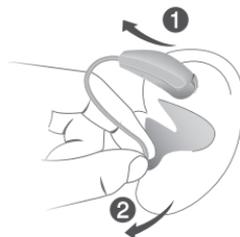

CAUTION

Risk of injury!

- ▶ In very rare cases the ear piece could remain in your ear when removing the hearing instrument. If this happens, have the ear piece removed by a medical professional.

Clean and dry your hearing aids after usage. Read more in section "Maintenance and care".

ADJUSTING THE VOLUME

Your hearing aids automatically adjust the volume to the listening situation.

- ▶ If you prefer manual volume adjustment, press the upper part of the rocker switch to increase the volume or press the lower part to decrease the volume.

Refer to section "Controls" for the setting of your rocker switch.

An optional signal can indicate the volume change. Once the maximum or minimum volume has been reached, you may hear an optional signal tone.

CHANGING THE HEARING PROGRAM

Depending on the listening situation, your hearing aids automatically adjust their sound.

Your hearing aids may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- ▶ To change the hearing program, press the rocker switch briefly.

Refer to section "Controls" for the setting of the rocker switch. Refer to section "Hearing programs" for a list of your hearing programs.

The default volume is applied.

FURTHER ADJUSTMENTS (OPTIONAL)

You can also use the hearing aid controls to change other features, for example, the level of the Tinnitus Function.

Refer to section "Controls" for the setting of your controls.

SPECIAL LISTENING SITUATIONS

ON THE PHONE

When you are on the phone, hold the telephone receiver slightly above your ear. The hearing aid and telephone receiver have to be aligned. Turn the receiver slightly so that the ear is not completely covered.

TELEPHONE PROGRAM

You may prefer a certain volume when using the phone. Ask your Hearing Care Professional to configure a telephone program.

- Switch to the telephone program whenever you are on the phone.

If a telephone program is configured for your hearing aids, it is listed in section "Hearing programs".

AUDIO STREAMING WITH IPHONE

Your hearing aids are **Made for iPhone** hearing aids. This means you can get phone calls and listen to music from your iPhone directly into your hearing aids.

For more information about compatible iOS devices, pairing, audio streaming and other useful functions, contact your Hearing Care Professional.

AUDIO STREAMING WITH ANDROID PHONE

If your mobile phone supports Audio Streaming for Hearing Aids (ASHA), you can get phone calls and listen to music from your mobile phone directly into your hearing aids.

For more information about compatible Android devices, pairing, audio streaming and other useful functions, contact your Hearing Care Professional.

BLUETOOTH

Your hearing aid is equipped with Bluetooth wireless technology to enable audio streaming from mobile phones or other compatible devices.

In an airplane, the use of Bluetooth functionality may be restricted, especially during take-off and landing. If this is the case, you can switch off Bluetooth wireless technology in your hearing aids via the smartphone app.

AUDIO INDUCTION LOOP

Some telephones and public places like theatres offer the audio signal (music and speech) via an audio induction loop. With this system, your hearing aids can directly receive the desired signal – without distracting noise in the environment.

You can usually recognize audio induction loop systems by certain signs.

Whenever you are in a place with an audio induction loop:

- ▶ Switch to the telecoil program.

If a telecoil program is configured for your hearing aids, it is listed in section “Hearing programs”.

MAINTENANCE AND CARE

To prevent damage it is important that you take care of your hearing aids and follow a few basic rules, which will soon become a part of your daily routine.

HEARING AIDS

CLEANING

Your hearing aids have a protective coating. However, if not cleaned regularly you may damage the hearing aids or cause personal injury.

- ▶ Clean your hearing aids daily with a soft, dry tissue.
- ▶ Never apply pressure while cleaning.
- ▶ Do not use alcoholic solvent on the charging contacts.
- ▶ To avoid corrosion, use a soft and dry tissue to wipe off sweat or liquid from the charging contacts or hearing aids before inserting them in the charger.
- ▶ Ask your Hearing Care Professional for recommended cleaning products, special care sets, or more information on how to keep your hearing aids in good condition.

When the hearing aids are dirtied by stubborn dirt, follow the steps below for thorough cleaning:

Note:

- Perform these steps with ThinTube/tube attached to avoid water coming into the hearing aids.

1. Rinse the hearing aids and rub off dirt under slow running tap water for 20 seconds.
2. Soak the hearing aids in warm water between 40 °C - 45 °C for 10 minutes.
3. Quick rinse the hearing aids again under slow running tap water for 10 seconds.
4. Allow the hearing aids to either air dry or dry the hearing aids with a soft and lint-free cloth. A recovery time of at least 2-6 hours is expected after these cleaning steps.

If water enters the ThinTube, detach the ThinTube from the hearing aid, blow it dry or air dry till it is completely dry.

Consult your Hearing Care Professional for further information on cleaning.

DRYING

You can use conventional products to dry your hearing aids. Ask your Hearing Care Professional for recommended drying products and for individual instructions on when to dry your hearing aids.

CAUTION

- ▶ To prevent damage to the internal waterproof membranes, do not use compressed air to blow into the microphone openings.

STORAGE

- **Short-term storage (up to several days):** Turn off your hearing aids via the rocker switch or place them in the powered charger.

The charger must be connected to the power supply. If the charger is not connected to power supply and you place your hearing aids in the charger, the hearing aids are **not** turned off.

Note that when you turn off the hearing aids via a remote control or smartphone app, the hearing aids are **not** turned off completely. They are in standby-mode and still draw some power.

- **Long-term storage (weeks, months, ...):** First, fully charge your hearing aids. Turn them off via the rocker switch before storing them. We recommend using drying products while storing the hearing aids.

At 6 month intervals you will have to charge the hearing aids to avoid an unrecoverable deep-discharge of the power cell. Deep-discharged power cells cannot be charged anymore and they have to be replaced. We recommend recharging more frequently than 6 month intervals.

Observe the storage conditions in section "Operating, transport and storage conditions".

EAR PIECES AND TUBES

Cerumen (ear wax) may accumulate on the ear pieces. This may affect sound quality.

- For hearing aids with a standard tube (ThinTube):

Clean the outside of the tubes daily with a tissue to remove any debris.

The tubes should be exchanged if you notice deterioration or discoloration.

Refrain from using cleaning products that contain alcohol or harsh chemicals as this may damage the tubes.

- For hearing aids with earhook:


Clean the ear pieces daily.

Ask your Hearing Care Professional to clean the tubes or to exchange the ear pieces and tubes as needed.

CLEANING EAR PIECES

Clean the ear pieces daily with a tissue to remove accumulated ear wax.

Replace the earpieces if they show signs of deterioration or cannot be cleaned satisfactorily.

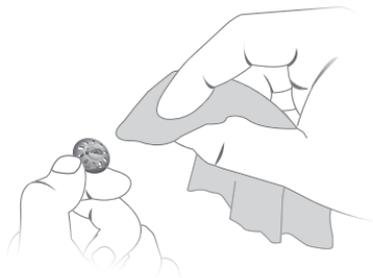EXCHANGING STANDARD EAR PIECES

- ▶ Pull off the used ear piece and plug on the new ear piece.

EXCHANGING STANDARD TUBES

- ▶ Unscrew the used tube from the hearing aid.

- ▶ Screw on the new tube.

PROFESSIONAL MAINTENANCE

Your Hearing Care Professional can perform a thorough professional cleaning and maintenance.

Custom-made ear pieces and wax filters should be exchanged as needed by your Hearing Care Professional.

Ask your Hearing Care Professional for your individual recommendation of maintenance intervals and for support.

IMPORTANT SAFETY INFORMATION

WARNING

A lithium-ion rechargeable battery is permanently built into your device. Incorrect use of the device can cause the lithium-ion rechargeable battery to burst.

Risk of injury, fire or explosion!

- ▶ Follow the safety instructions for lithium-ion rechargeable battery in this section.

WARNING

Risk of electric shock!

- ▶ Connect the charger only with a USB power supply which complies with IEC 60950-1 and/or IEC 62368-1.

WARNING

- ▶ Observe the operating conditions.
- ▶ Pay special attention to protect your device from extreme heat (e.g. fire, microwave oven, induction oven or other high induction fields) and sunlight.
- ▶ Charge your hearing instruments only with an approved charger. Ask your Hearing Care Professional for support.
- ▶ Do not disassemble the device.
- ▶ Do not use the device in explosive atmospheres.
- ▶ Do not use devices that are deformed or obviously damaged.
- ▶ Do not use the device when its performance after charging reduces considerably. Return the device to your Hearing Care Professional.
- ▶ Do not dispose of the device in fire or water.

WARNING

- ▶ Replacement and repair have to be performed by a professional. Therefore:
- ▶ Do not repair or replace the lithium-ion rechargeable battery yourself.
- ▶ In the unlikely event of a lithium-ion rechargeable battery bursting or exploding, keep all combustible materials away.

In any of the above cases, return your hearing aids to the Hearing Care Professional.

FURTHER INFORMATION

SAFETY INFORMATION

For more safety information, refer to the safety manual supplied with the device.

ACCESSORIES

You can use the smartphone app for convenient remote control of your hearing aids. The app also allows you to deactivate signal tones, to create individual sound profiles and many more functions.

You can also use a remote control.

Consult your Hearing Care Professional for further information.

OPERATING, TRANSPORT, AND STORAGE CONDITIONS

The hearing aids operate in the following environmental conditions (also valid in between uses):

Operating conditions	Discharging	Charging
Temperature	0 to 50 °C (32 to 122 °F)	10 to 35 °C (50 to 95 °F)
Relative humidity	5 to 93 %	5 to 93 %
Atmospheric pressure	700 to 1060 hPa	700 to 1060 hPa

During extended periods of transport and storage, please observe the following conditions:

	Storage	Transport
Temperature (recommended)	10 to 25 °C (50 to 77 °F)	-20 to 60 °C (-4 to 140 °F)
Temperature (maximum range)	10 to 40 °C (50 to 104 °F)	-20 to 60 °C (-4 to 140 °F)
Relative humidity (recommended)	20 to 80 %	5 to 90 %
Relative humidity (maximum range)	10 to 80 %	5 to 90 %
Atmospheric pressure	700 to 1060 hPa	700 to 1060 hPa

Note that you have to charge the hearing aids at least every 6 months. We recommend to charge the hearing aids every 3 months. For details refer to section "Maintenance and care".

For other parts, such as the charger, other conditions may apply.

WARNING

Risk of injury, fire or explosion during shipment.

- ▶ Ship power cells and hearing aids according to local regulations.

DISPOSAL INFORMATION

- ▶ To avoid environmental pollution, do not throw power cells or hearing aids into household trash.
- ▶ Recycle or dispose of power cells or hearing aids according to local regulations.

SYMBOLS USED IN THIS DOCUMENT

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

Made for

iPhone | iPad | iPod

"Made for iPhone", "Made for iPad", and "Made for iPod" mean that an electronic accessory has been designed to connect specifically to iPhone, iPad, or iPod, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone, iPad, or iPod may affect wireless performance.

TROUBLESHOOTING

Problem and possible solutions

Sound is weak.

- Increase the volume.
- Charge the hearing aid.
- Clean or exchange the ear piece.

Hearing aid emits whistling sounds.

- Reinsert the ear piece until it fits securely.
- Decrease the volume.
- Clean or exchange the ear piece.

Sound is distorted.

- Decrease the volume.
- Charge the hearing aid.
- Clean or exchange the ear piece.

Hearing aid emits signal tones.

- Charge the hearing aid.

Hearing aid does not work.

- Turn the hearing aid on.
- Charge the hearing aid. Observe also the status LEDs on the charger.

Consult your Hearing Care Professional if you encounter further problems.

COUNTRY-SPECIFIC INFORMATION

USA AND CANADA

Certification information for hearing aids with wireless functionality:

BiCore B-Li M Rugged

- HVIN: RFM008
- Contains FCC ID: 2AXDT-RFM008
- Contains IC: 26428-RFM008

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© 02.2023, WSAUD A/S. All rights reserved.
WSAUD A/S