

REXTON REACH INOX-CIC LI INSTANT-FIT 3.0 EARMOLD



The Reach inoX-CIC Li is an innovative instant-fit ready-to-wear CIC that offers an immediate fit with immediate impact.

It can be instantly-fit with EarWear 3.0 eartips and sleeves, and our new inoX Instant-Fit 3.0 Earmolds.

We still recommend eartips and sleeves as the primary first choice fitting option.

Instant Fit 3.0 earmolds are available in clear silicone (soft) and clear acrylic (hard) materials.



CASES WHERE AN INSTANT-FIT 3.0 EARMOLD MAY BE BENEFICIAL

- Better retention if inoX fit is not secure and tends to work out with eartips/sleeves.
- Better fit or comfort if eartips/sleeves are not optimal.
- A previous CIC wearer wants to upgrade to the newest rechargeable CIC technology and prefers the fit/feel of a custom earmold.

We still recommend first trying the Reach inoX with eartips/sleeves as they will feel less occluding.

- The hearing loss may be pushing the fitting range limit, and you want to close the fitting as much as possible to reduce the chance for feedback.

We still encourage trying eartips/sleeves first before ordering the instant-fit 3.0 earmold.

REXTON

BEST PRACTICE CONSIDERATIONS FOR INOX INSTANT-FIT 3.0 EARMOLDS

- Good impressions are important in creating properly fitting earmolds.
- Ear canal shape/size will determine how well the earmold fits.
 - If the Reach inoX protrudes from the ear with eartips or sleeves, it will also protrude with an earmold.
 - If the member would not typically be a candidate for a CIC hearing aid, they will likely not be a candidate for an instant-fit 3.0 earmold.
- Using an instant-fit 3.0 earmold with the Reach inoX hearing aid will create a fitting which is more closed or occluding than eartips and sleeves.

ORDERING REACH INOX INSTANT-FIT EARMOLD 3.0

Orders can be placed online using a scan on file (impression on file from a previous earmold/custom order) or with Otoscan. You can also print an online order confirmation and send with impressions.

ONLINE ORDER

Place order under Earmolds > Instant Fit 3.0 Earmold

- **Option 1:** Print order confirmation and mail with impressions.
- **Option 2:** Use previous scan on file. Be sure to note the serial number(s) of previous earmold/custom device in Comments on the Review & Payments page.
- **Option 3:** Attach Otoscan: include Otoscan ID in Comments on the Review & Payments page.

PDF ORDER FORM

Fill out, print and mail with impressions. Located in your Managed bookmarks and at [Forms - Rexton](#).

HEARING DEVICES ▾ EARMOLDS ▾ ACCESSORIES AND PARTS ▾ SERVICES ▾ WARRANTIES ▾ MARKETING LIBRARY ▾ SOFTWARE UPDATES ▾ MY ORDERS ▾

Instant Fit 3.0 Earmold

Right Side: Instant Fit 3.0 Earmold 1

Left Side: Instant Fit 3.0 Earmold 1

Patient Reference: * Syde Patient ID
Maximum number of characters: 20 (4 remaining)

Default warranty: 3 Months

REXTON My Store My

HEARING DEVICES ▾ EARMOLDS ▾ ACCESSORIES AND PARTS ▾ SERVICES ▾ WARRANTIES ▾ MARKETING LIBRARY ▾ SOFTWARE UPDATES ▾ MY ORDERS ▾

Earmolds

Extended Receiver Unit

- Swim Protection Mold
- Standard Mold
- Thintube 3.0 Earmold
- RIC 3.0 Earmold
- RIC 2.0 Earmold
- Protection Mold
- Instant Fit 3.0 Earmold

ERU HP Receiver 3.0

ERU HP Receiver 2.0

REXTON

1 Shipping 2 Notes 3 Review & Payments

P.O. Number: * 0000000000

Fit By Name: Enter Fit By Name

Contact Information (Optional): Contact Name Phone Number (Optional): Phone Number

Comment (Optional): Enter Serial number of previous custom product scan on file or Otoscan ID here