


# PROGRAMMING TELECOIL

Most Rexton hearing aids are equipped with a telecoil that can be activated through the Connexx fitting software. Telecoil technology allows the hearing aid to connect directly to compatible looped sound systems, giving direct hearing access to small and large public areas.

For a list of looped systems in your area, visit: <https://time2loopamerica.com/loop-locator/>


To activate a telecoil program:

1. Locate the program handling tab.
2. Add a program in the next available program slot by clicking the drop-down arrow.

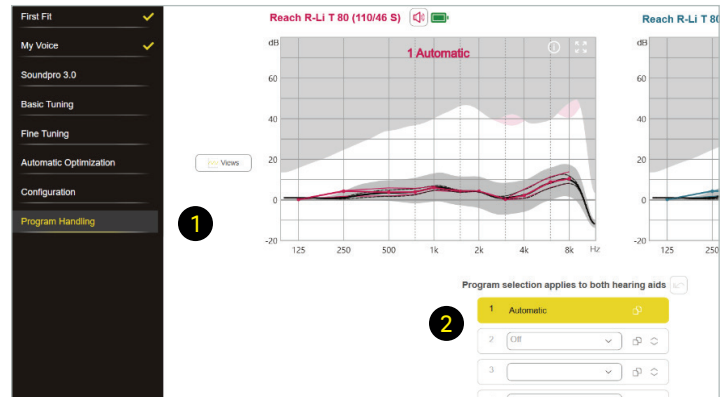


3. Select “Induction Loop,” and then select the desired telecoil setting :

- **Telecoil (mT):** Telecoil + low mic mix  
- Default is -12dB mic level mix
- **Telecoil (T):** Telecoil only, mics disabled

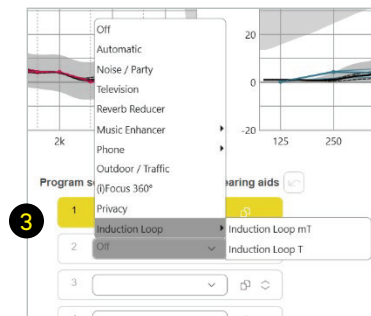

4. You may receive a notification requiring activation of rocker switch for program changes. Then select OK.

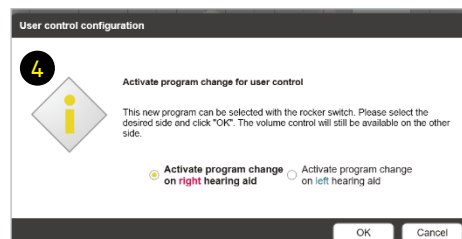

5. Adjust gain and microphone mixing settings if needed in Fine Tuning tab. Mix settings are found under Noise Reduction/Microphone.

The member will activate their telecoil program by pressing the program change button on their hearing aids or in the Rexton app.

