


REXTON

PAIRING TO iPhone® FOR STREAMING REACH R PLUS

Bluetooth® Classic connects your hearing aids for streaming calls and audio. Pairing is performed through your phone's streaming audio and headset mode Bluetooth menu.

STEP 1

Turn your phone's Bluetooth on.  Pair your hearing aids to the Rexton App (download from the App Store). **Note:** if your hearing aids are already paired to another device nearby, turn off the Bluetooth on that device before starting the pairing process.

STEP 2

Open the phone **Settings** menu. Tap **Bluetooth** to open the Bluetooth menu.



STEP 3



Turn your hearing aids off and on by placing them into the charger for 5 seconds then taking them back out.

The green LED indicator light will illuminate for 6 seconds; then the hearing aids are ready to be paired to your phone. The hearing aids will remain in pairing mode for 3 minutes.



STEP 4

Your hearing aids will appear under **Other Devices**. Tap the **hearing aid name** to pair them. Your hearing aids will now be listed under **My Devices** and are **connected**.

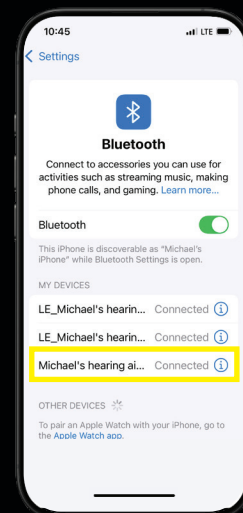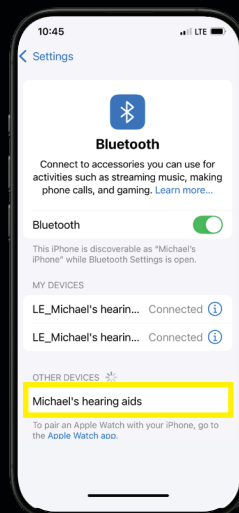

EASILY SWITCH BETWEEN PAIRED BLUETOOTH DEVICES

For example: You are currently connected to iPhone and wish to connect to your PC:

1. Switch off Bluetooth on iPhone.
2. Hearing aid looks for connection; go to PC and connect to hearing aid.
3. Once hearing aid is connected to PC, switch on iPhone Bluetooth again.

Please note:

In your Bluetooth menu, the name of your headset and streaming connection will match the name assigned to your hearing aids when initially programmed by your Hearing Care Provider.

STILL NEED HELP PAIRING YOUR iPhone OR ANDROID?

Call Rexton Consumer Line:
1-866-212-0610
Monday-Friday 8am to 8pm EST

Distributed by:
WS Audiology US Inc.
101 Wood Avenue South, 7th Floor
Iselin, New Jersey 08830



The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc. and any use of such marks by WSAUD A/S is under license. Other trademarks and trade names are those of their respective owners. Apple and iPhone are trademarks of Apple Inc. App Store is a service mark of Apple Inc. Google Play is a trademark of Google Inc. The Android name, the logo, and other trademarks are property of Google LLC.

Copyright © 2025 Rexton is a registered trademark of WSAUD A/S.


10248779 | RX-23543 | 12.2025

REXTON

PAIRING TO ANDROID™ FOR STREAMING REACH R PLUS


Bluetooth® Classic connects your hearing aids for streaming calls and audio. Pairing is performed through your phone's streaming audio and headset mode Bluetooth menu.

STEP 1

Turn your phone's Bluetooth on.  Pair your hearing aids to the Rexton App (download from Google Play). **Note:** if your hearing aids are already paired to another device nearby, turn off the Bluetooth on that device before starting the pairing process.

STEP 2

Open the phone **Settings** menu. Tap **Connections**. Tap **Bluetooth** or **Settings** to open the Bluetooth menu.

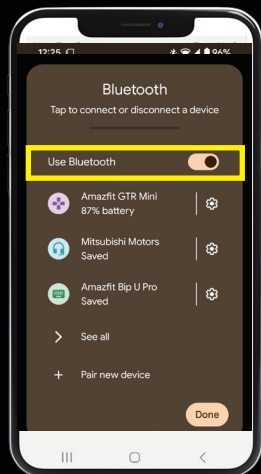

STEP 3


Turn your hearing aids off and on by placing them into the charger for 5 seconds then taking them back out.

The green LED indicator light will illuminate for 6 seconds; then the hearing aids are ready to be paired to your phone. The hearing aids will remain in pairing mode for 3 minutes.




STEP 4

Your hearing aids will appear under **Available Devices**. Tap the **hearing aid name** to pair them.

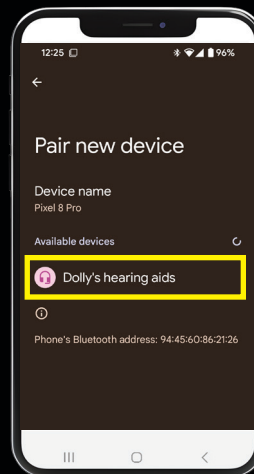

STEP 5

A pairing request will appear. Tap **Pair** to complete the pairing request.

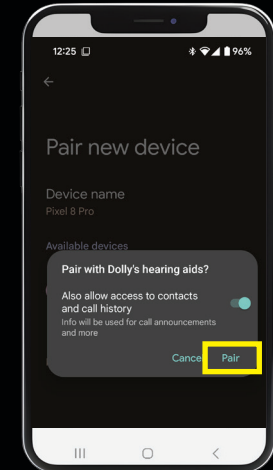

STEP 6

The hearing aids will appear under **Paired devices** or **Media devices** with a **Connected** status.

